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Restart of Tandem t:slim X2™ insulin pump

Please read the full instruction before restarting you insulin pump!

Disconnect the pump from your body. Confirm that at least 50 units of insulin remains in the cartridge before restarting the pump. Note that when the pump has been switched off, you must load the cartridge again (you can use the “old” cartridge if the insulin is still ok to use) and fill the tubing with at least 10.2 units. You will also need to reselect the language after restarting the pump, but all other settings remain. If your sensor session is active, do not stop the sensor before restarting the Tandem t:slim X2 insulin pump. If you do not use the Dexcom G6 CGM with the Tandem t:slim X2 insulin pump, you can go ahead and skip step 6.

1. Connect the pump to a power source.
2. Press and hold the start-/quick bolus button for at least 30 seconds even if the pump lights up or beeps during this process. Continue to hold down the button. The pump will beep and then switch off. Disconnect the pump from the power source and confirm that the pump is switched off by pressing the start-/quick bolus button, the screen should not light up.
3. The pump should now rest without being connected to a power source for 20 minutes.
4. Reconnect the pump to a power source. The screen will automatically light up – just wait.
Observe: Do not hold down the Start – /Quick bolus button to start the pump.
5. Select your language and redo the load process for the cartridge. Fill the tubing with at least 10.2 units of insulin.
6. If you are using your CGM, the CGM trend graph will temporarily be paused. Restart the sensor session. Re-enter the sensor code in the pump. If you do not remember your sensor code, you can find it in your pump under Options/History/CGM history/Complete, select the day you started the sensor session and look for “sensor code entered”. The four-digit code is on the right below the time. There will not be a new warm-up period, but it may take a while before you get your CGM trend graph back.
7. Keep in mind that the insulin on board is reset when the pump is restarted.