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## Restart of Tandem t:slim X2™ insulin pump

**Please read the full instruction before restarting you insulin pump!**

**Disconnect the pump** from your body. Confirm that at least 50 units of insulin remains in the cartridge before restarting the pump. Note that when the pump has been switched off, you must load the cartridge again (you can use the “old” cartridge if the insulin is still ok to use) and fill the tubing with at least 10.2 units. You will also need to reselect the language after restarting the pump, but all other settings remain. If you do not have an active sensor session, you can go ahead and skip step 6.

1. Connect the pump to a power source.
2. Press and hold the start-/quick bolus button for at least 30 seconds even if the pump lights up, vibrates or beeps during this process. Continue to hold down the button even if the screen goes dark. The pump will beep and then switch off. Disconnect the pump from the power source and confirm that the pump is switched off by pressing the start-/quick bolus button, the screen should not light up.
3. The pump should now rest without being connected to a power source for **20 minutes**.
4. Reconnect the pump to a power source. The screen will automatically light up – just wait.  
Observe: Do not hold down the Start – /Quick bolus button to start the pump.
5. Select your language and redo the load process for the cartridge.
6. Your CGM trend graph will temporarily be paused. You will need to restart the sensor session in the pump.  
**If you are using Dexcom G7:** Go to Options --> My CGM --> Start G7 sensor. Press the blue checkmark to reconnect to your current sensor session.  
**If you are using Dexcom G6:** Go to Options --> My CGM --> Start G6 sensor. You may select SKIP when the pump asks for the Sensor Code when you reconnect to your current sensor session.
7. Keep in mind that the Insulin On Board (IOB) is reset when the pump is restarted.