

Waxaan halkan u joognaa adiga

Rubin Medical, waxaan dooneynaa in dadka isticmaalayaasha ah ee adiga oo kale ah ay dareemaan ammaan iyo qanacsanaan ku aadan nidaamkaaga bamka insuliinta.

Webseedkeena, waxaad ka heli doontaa jawaabaha su'aalaha inta badan la isweydiiyo ee ku saabsan alaabaheena, sheekooyinka isticmaalayaasha, tallooyinka, iyo macluumaadka ku saabsan nooca 1 aad ee cudurka macaanka.

Waxaa sidoo kale jira ikhtiyaar ah in lagu soo gudbiyo **wixii su'aalo ama cabashooyin ah oo ku saabsan dareemayaasha oonleenka.**

Haddii aad qabto wax su'aalo ah, fadlan wac adeeggeena taageerada macaamiisha ama noogu soo dir iimayl **info@rubinmedical.se**. Adeeggeena taageerada macaamiisha ayaa hadda sidoo kale diyaar ah fiidkii Talaadada iyo Arbacada si uu kaaga caawiyo inaad ka jawaabto su'aalahaaga ku saabsan qalabkeena gargaarka cudurka macaanka. Dabcaan, wali waad la xiriiri kartaa adeeggeena taageerada farsamada bamka wakhti kasta.

Waxaan halkan joognaa mar kasta oo aad noo baahato.



Si aad ula socoto wararkii ugu dambeeyay, ku soo biir bulshadeena ood ka helayso

[facebook.com/rubinmedical](https://www.facebook.com/rubinmedical)
[instagram.com/rubin.medical](https://www.instagram.com/rubin.medical)

Taageerada macaamiisha:

Talaado iyo Arbacada, 8:00 subaxnimo ilaa 7:00 fiidnimo

Maalmaha kale ee shaqada: 8:00 subaxnimo ilaa

4:00 galabnimo

040-15 54 80

Taageerada farsamada bamka (dhammaan waqtiyada kale):

044-330 220


www.rubinmedical.se



© 2025 Rubin Medical AB. Dhammaan Xuquuqaha Way Dhowrsan Yihiin Tandem Diabetes Care waa astaan ganacsi oo diiwaangashan, t:slim X2™, Basal-IQ™, Control-IQ™, AutoSoft™, VariSoft™, TruSteel™, aabka moobeelka t:slim™, iyo t:simulator™ waa astaamo ganacsi oo ay leedahay Tandem Diabetes Care Inc. Dexcom, Dexcom G6, iyo Dexcom G7 waa astaamo ganacsi oo diiwaangashan oo ay leedahay Dexcom Inc. oo ku yaala Mareykanka iyo/ama waddamo kale. Astaamaha iyo calaamadaha Bluetooth® waa calaamada ganacsi oo diiwaangashan oo ay leedahay Bluetooth SIG, Inc. isticmaal kasta oo astaantaan ah oo ay sameyso Tandem Diabetes Care, Inc. waa mid shati lagu bixiyay. Dhammaan astaamaha ganacsiga ee kale ee dhinac saddexaad waxay leeyihiin mulkiilayaashooda.



Kahor iyo kadib markaad billoowdo isticmaalida bamkaaga

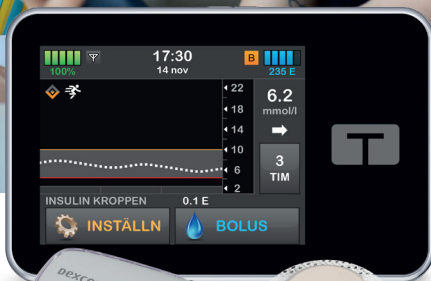

Bamka insuliinta Tandem t:slim X2™ oo wata
Dexcom CGM iyo aabka moobeelka Tandem t:slim™

Hambalyo inaad heshay nidaamkaaga cusub ee bamka insuliinta

Markaad billoowdo isticmaalida nidaamkaaga bamka insuliinta, waa muhiim inaad dareento ammaan iyo raaxo ku aadan isticmaalkeeda iyo inaad leedahay faham aasaasi ah oo ku saabsan sida saxda ah ee loo isticmaalo. Waxaa jira macluumaad badan oo aad qaadan karto marka aad billoowdo isticmaalida bamka, sidaa darteed waxaan si xooggan kugula talineynaa inaad akhriso hagahaan oo aad daawato muuqaalada tilmaamaha ah ee aan soo jeedineyno si ay kaaga caawiyaan diyaarinta.

Waxaad u baahan tahay inaad billoowdo isticmaalida bamkaaga

Markaad billoowdo isticmaalka bamkaaga, waxaad u baahan doontaa inaad haysato dhammaan waxyaabaha aad u isticmaali doonto daaweyntaada:

- Bamka insuliintaaga
- Qalabka faleebada
- Aalada Cartridge-ka
- Insuliin
- Dib u buuxinta saliingaha
- Boorso leh agabyo
- Taleefanka gacanta oo si buuxda loo jaajar gareeyay



- Dexcom CGM haddii tani ay ku jirto daaweyntaada




- › Alaabahaan waxaa badanaa la bixiyaa kahor, laakiin waxaa sidoo kale lagugu siin karaa rugta caafimaadka marka aad billoowdo isticmaalida bamkaaga. La xirii rugtaada caafimaadka haddii aadan hubin ama aad dooneyso inaad dalbato qalab dheeraad ah.

Sida loo diyaariyo

1. Waa muhiim inaad iska diiwaangeliso isticmaale ahaan si Rubin Medical ay kuu siiso macluumaad muhiim ah. Habka ugu fudud ee arrintaan lagu sameyn karo waa adigoo iskaan gareynaya koodhka QR-ka ee dhanka midig ama gujinaya halkan.



2. Soo dejiso **aabka tusaalaha ah ahna bilaashka ah ee t:simulator™** si aad u barato sifooyinka bamka insuliintaada cusub. Aabka dhexdiisa, waxaad ku arki kartaa sida ay u shaqeeyaan labadaba teknoolajiyada Control-IQ™ iyo tignoolajiyada Basal-IQ™.

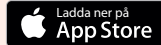

Dooro "Tilmaamaha Shaqada" si aad u hesho dulmar guud oo ku saabsan sifooyinka iyo xulashooyinka liiska. Marka aad arrintaan eegto, waxaan kugula talinaynaa inaad demiso "Tilmaamaha Shaqada". Si aad u furto shaashadda t:simulator, waxaad u baahan tahay inaad gujiso 1, 2, iyo 3.



Qaamuuska erayada

Waxaa faa'iido yeelan karta inaad ogaato erayada caadiga ah ee loo isticmaalo daaweynta bamka insuliinta si ay kaaga caawiyaan inaad raacdo tilmaamaha marka aad billoowdo isticmaalka bamkaaga.

AutoBolus

AutoBolus waa astaan ku jirta Tandem t:slim X2™ taasoo bixisa bolus dheeraad ah (hal qiyaas) oo insuliin ah haddii heerka sonkorta dhiiggaagu la saadaaliyo inay ka sarreyso 10.0 mmol/L 30 daqiiqo gudahood. Tan waxaa loogu talagalay inay si toos ah u hagaajiso sonkorta dhiiggaaga ka dhigto 6.1 mmol/L oo ay ka hortagto inay aad u sarreyso. Astaantaan waxaa la shidi karaa hal mar saacaddii waxaana la isticmaali karaa iyadoo lala kaashanayo qiyaasta asaasiga ah, taas oo la hagaajiyayo shan daqiiqo kasta.

Kuurada Xakameynta (Basal)

Kuurada xakameynta waxay si tartiib ah oo joogto ah u bixisaa insuliin, taasoo heerarka sonkorta dhiiggaaga ka dhigaysa mid deggan inta u dhaxaysa cuntooyinka iyo inta aad hurudo. Waxaa lagu cabiraa halbeega insuliin saacaddiiba.

BG

BG waxay u taagan tahay gulukooska dhiigga (sonkorta dhiigga), oo ah heerka gulukooska dhiiggaaga. Waxaa lagu cabbiraa mmol/L.

Bolus

Bolus waa hal qiyaas oo insuliin ah oo si dhakhso ah u shaqeyso oo badanaa la bixiyo si ay kaaga caawiso inaad cunto cunno ama aad hagaajiso heerka sonkorta dhiigga oo sareysa.

Kaarboohaydareet

Kaarboohaydareetku waa sonkor iyo istaaj oo uu jirkaagu u kala jabiyo gulukoos oo uuna u isticmaalo ilo tamar. Waxaa lagu cabbiraa garaam.

Bolus-ka sixitaanka

Bolus sixitaanka ah ayaa la bixiyaa si loo saxo heerka sonkorta dhiigga ee sareysa.

Qodobka Sixitaanka

Qodobka sixitaanka waa inta sonkorta dhiiggaaga lagu dhimi doono 1 unug oo insuliin ah. Tan waxaa sidoo kale loo yaqaan saameynta insuliinta (insulin sensitivity factor, ISF).

Bolus la dheereeyey

Bolus-ka la dheereeyay waa bolus (qiyaas) la bixiyo muddo dheer. Badanaa waxaa loo isticmaalaa in lagu caawiyo habaynta cuntada oo waqti dheer qaadata in calooshu dheefshiido. Marka bolus la dheereeyay la keeno bamka, waxaad doorataa SII DAA HADDA (DELIVER NOW) si aad isla markiiba u bixiso qaar ka mid ah insuliinta, inta soo hartayna muddo dheer.

Garaam

Garaam waa halbeegga cabbirka karbohaydareetyada.

Muddada insuliinta

Muddada insuliintu waa muddada ay insuliintu shaqeyneyso oo laga heli karo jirkaaga ka dib marka la soo saaro bolus. Tani waxay saameyn ku leedahay xisaabinta Insulin Boodhka (IOB).

Insuliinta Boodhka (Insulin on Board, IOB)

IOB waa insuliinta wali ka shaqeysa jirkaaga (sidaas darteedna awoodda aad ku sii wadi karto hoos u dhigista heerka sonkorta dhiiggaaga) ka dib marka la soo saaro bolus.

Saamiga Insuliinta ilaa-Karbohaydareetka

Saamiga karbohaydareetka waa tirada garaam ee karbohaydareetyada oo ay ku jirto 1 unug oo insuliin ah. Tan waxaa sidoo kale loo yaqaan ICR.

Lasoo gelinaayo

Soo gelinta waxaa loola jeedaa habka looga saarayo aalada cartridge-ka iyo buuxinta iyo gelinta mid cusub, iyadoo lala kaashanayo marin cusub oo la geliyo.

Cirbad

Cirbaddu waa qayb ka mid ah qalabka wax lagu duro. Waxaa la geliyaa maqaarka hoostiisa si jirka loo geliyo insuliin.

Xogta borofaylka

Xogtaada borofaylka waa hab go'aaminaya qiyaasaha insuliinta basal iyo bolus ee la bixiyo muddo cayiman gudahood muddo 24-saacadood ah.

Bolus Degdeg ah

Kani waa sifa kuu oggolaanaysa inaad keento qiyaas bolus ah adigoo si fudud u riixaya badhan ku yaala dusha sare ee bamka insuliinta.

Bartilmaameed BG

Bartilmaameedka BG waa heerka bartilmaameedka BG (sonkorta dhiigga) - tirada saxda ah, ee maaha kala duwanaansho. Marka qiimaha BG lagu geliyo bamka, qiyaasta insuliinta ee la xisaabiyay ayaa kor ama hoos loo hagaajiyaa si loo gaaro heerka bartilmaameedka ah ee 6.1 mmol/L.

Unugyada

Unugyada waa cutubyada cabbirka ee lagu go'aamiyo xaddiga insuliinta.

Baro sida loo isticmaalo qalabkaaga

Hoosta [Tababarka iyo Macluumaadka](#) ee bogga internetka ee Rubin Medical, waxaad ka heli doontaa agabyo sida **hagayaal degdeg ah** oo ku saabsan nidaamka bamka insuliinta ee Tandem t:slim X2™ oo leh teknoolojiyadda Control-IQ™ ama teknoolojiyadda Basal-IQ™ iyo Dexcom CGM iyo tilmaamo ku saabsan sida loo bilaabo aabka moobaylka ee Tandem t:slim™ ee taleefankaaga casriga ah. Waxa kale oo aad ka heli doontaa **muuqaalo tilmaamo ah** oo ku saabsan sida loo adeegsado qalabka daawada iyo CGM iyo sida loo buuxiyo aaladaha cartridges.



Muuqaalo tilmaamo si ay kaaga caawiyaan inaad billoowdo:

Soo rarista aalada cartridge cusub



Isticmaalida agabka daaweynta AutoSoft™ 90



Isticmaalida agabka daaweynta AutoSoft™ 30



Isticmaalida agabka daaweynta VariSoft™



Isticmaalida agabka daaweynta ee TruSteel™



Adeegsiga Dexcom G6 CGM



Isticmaalida Dexcom G7 CGM kuna xirista gacantaada sare



Isticmaalida Dexcom G7 CGM kuna xirista gumaarkaaga



Bixinta qiyaas bolus ah



Bixinta qiyaas bolus ah oo dheeraad ah



Macluumaad ku saabsan aabka moobeelka ee Tandem t:slim



Tababar oonleen ah oo loogu talagalay aabka moobeelka ee Tandem t:slim



Kaamirada taleefankaaga gacanta ku sawir lambarrada QR-ka.

Farriimaha khaladaadka caadiga ah:

Waa kuwaan waxa la sameeyo marka qaylo-dhaanta bamka ay dhawaaqdo.

Digniin ka baxsan masaafada (14C)

Sababta: Dareemaha iyo bamka ma xiriirayaan.

Waxa la sameeyo: Bamka dhig meel u dhow dareemaha, xiriirkuna si toos ah ayuu u soo noqon doonaa.

CGM lama heli karo (48T)

— — — waxaa lagu muujiyaa shaashadda bamka oo aad caadiyan ku aragto tirada dareemayaasha.

Sababta: Fiilada dareemayaasha hadda ma awoodo inay akhriisi heerka gulukooska.

Waxa la sameeyo: Waxaa laga yaabaa inay ku caawiso cabista biyaha/dareeraha iyo ku xirista dareemaha cajalad si looga hortago inay daataan. Tani waxay dhici kartaa habeenkii haddii aad ku jifto dareemaha, tusaale ahaan, maadaama tani ay carqaladeynayso wareegga ku wareegsan fiilada dareemaha. Waxaad arki kartaa in GCM aan laga heli karin qalooqa isbadelka ee shaashadda marka ay jiraan dhibco maqan oo ku jira qalooqa. Haddii mid ka mid ah tallooyinka kor ku xusan uusan caawin oo aysan jirin tiro dareemayaal ah muddo saddex saacadood ah, fadlan la xiriir taagerada macaamiisheena inta lagu jiro maalinta si aad u hesho xalal kale oo lagu xalliyo dhibaataada.

Digniin sare oo Control-IQ ah

Laba gariir, ka dibna laba gariir/ballaar shan daqiiqo kasta ilaa aad xaqiijiso ama qiimaha BG-gaagu hoos uga dhaco heerka digniinta, ka dibna saacad kasta oo kale haddii dhibaataadu sii socoto.

Sababta: Akhrinta gulukooska dareemaha ee ugu dambaysay waxay gaartaa 11.1 mmol/L ama ka badan, nidaamkuna wuxuu saadaalinayaa in akhrintu aysan hoos u dhici doonin 30ka daqiiqo ee soo socda.


Waxa la sameeyo: Hubi aalada cartridge, tuubada, iyo goobta lagu shubay oo tijaabi BG-gaaga. Ula dhaqan heerkaaga BG ee sareeya sida loogu baahdo. Riix si aad u xaqiijiso.

Digniin hoose oo Control-IQ

Laba gariir, ka dibna laba gariir/ballaar shantii daqiiqoba mar ilaa aad xaqiijiso ama heerka BG-gaagu uu ka sarreeyo heerka digniinta.





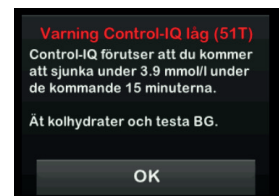

Sababta: Nidaamku wuxuu saadaalinayaa in akhrinta gulukooska dareemahaagu ay hoos ugu dhici doonto 3.9 mmol/L, ama ka hoosaysa 4.4 mmol/L haddii dejinta "Hawlgalka Jimicsiga" la hawlgeliyo, 15 ka daqiiqo ee soo socda.

Waxa la sameeyo: Cun karbohaydareetka oo tijaabi BG-gaaga. Riix si aad u xaqiijiso.



Astaamaha ku yaala bamka insuliinta iyo shaashada guriga ee aabka moobeelka

Midabka astaanta dheemanka ee ku taala bamka insuliinta iyo shaashadda aabka moobeelka ayaa muujinaya sixitaanka ama hagaajinta qiyaasta insuliinta ee nidaamku hadda sameynayo. Shan daqiiqo kasta, bamka insuliinta Tandem t:slim X2™ oo wata teknoolajiyadda Control-IQ™ ayaa go'aan cusub ka gaaraya in la bixiyo qiyaas iyo sida loo bixiyo.

Astaamaha:	Sharaxaada:
  Kuurada	AutoBolus (qiyaas bolus ah oo ootomaatig ah) ayaa la bixiyaa haddii tirada dareemaha la saadaaliyo inuu noqon doono 10.0 mmol/L ama ka badan 30 daqiiqo gudahood (kuma jiraan qaabka "Hawlgalka Hurdada")
 B Kordhinta	Qiyaasta asaasiga ah way kordheysaa haddii qiimaha dareemaha la saadaaliyo inuu gaaro ama ka bato 8.9mmol/L 30 daqiiqo gudahood
 B Joogteynta	Qiyaasta aasaasiga ah ee la borogaraamiyay lagu sameeyay ayaa la bixiyaa marka qiimaha dareemaha aan la saadaalin karin inuu hoos uga dhaco 6.25 mmol/L ama uu ka kordho 8.9 mmol/L 30 daqiiqo gudahood.
 B Hoos u dhaca	Qiyaasta asaasiga ah ayaa hoos u dhacda haddii tirada dareemaha la saadaaliyo inuu hoos ugu dhaco 6.25 mmol/L ama ka hooseeyo 30 daqiiqo gudahood
 0 Joojinta	Qiyaasta aasaasiga ah waa la joojiyaa haddii tirada dareemaha la saadaaliyo inuu hoos ugu dhaco 3.9 mmol/L ama ka hooseeyo 30 daqiiqo gudahood.

Talo:

Weydii kalkaalisadaada sida loo dalbado dareemayaal cusub iyo agab kale oo u qor si xasuusin ah:

Fadlan halkan saxiix si aad u xaqiijiso inaad akhrisay buug-yaraha oo aad daawatay muuqaalada ku jira bogga interneetka oo aad u tusto kalkaaliyahaaga cudurka macaanka marka aad billoowdo isticmaalka bamkaaga.

Taariikhda:

Saxiixa:

Waa fiican tahay in la ogaado

- 1. Cusub!** Aabka moobeelka ee Tandem t:slim™ hadda waa la heli karaa - codsi taleefanka gacanta ah oo aad u isticmaali karto inaad si toos ah uga shido qiyaasaha bolus taleefankaaga.* Taas macnaheedu waa inaad si fudud oo qarsoodi ah u bixin karto qiyaasaha, adigoon u baahnayn inaad ka saarto bamka insuliintaada. Aabku wuxuu ku siinayaa dulmar cad oo ku saabsan qiyaasahaaga basal iyo bolus, insuliinta firfircoon, karbohaydareetyada, sitinada bamka, iyo xogta CGM - dhammaantood hal meel ayay ku jiraan qalabkaaga moobeelka. Tani waxay kuu sahlaysaa inaad la socoto daawayntaada waxayna ku siinaysaa xorriyad dheeraad ah oo ah nolol maalmeedkaaga. Dhammaan xogta daaweynta waxaa si toos ah iyo si bilaawayir ah loogu soo shubayaa Tandem Source™, adeeg ku saleysan oonleen oo aad ku falanqeyn karto xogtaada adoo isticmaalaya warbixinno si fudud loo akhrin karo.
- 2.** Ma jeclaan lahayd inaad ku aragto cadadka gulukooskaaga saacaddaada casriga ah? Kala soco qiimayaashaada CGM waqtiga dhabta ah ee saacaddaada casriga ah iyada oo loo marayo aabka **Dexcom G7**, iyadoo ku xiran dareemaha aad isticmaaleyso. Waxa kale oo aad awoodi doontaa inaad tiradaada ku aragto aabka taleefankaaga casriga ah. Haddii daryeel bixiye ama qaraabo ay sidoo kale rabaan inay la socdaan tirada, waxay sidaas ku samayn karaan iyagoo soo dejinaya **Aabka La socoshada Dexcom** taleefankooda. Tani waxay suurtagal ka dhigaysaa in tirada CGM uu si toos ah ula wadaago aabka Dexcom Aabka La socoshada.
- 3. T1D** waa aabka macluumaadka si fudud loo heli karo ee ku saabsan cudurka macaanka nooca 1 aad iyo qalab aad u fiican oo lagu bilaabi karo hababka maaraynta cudurka macaanka sida tirinta karbohaydareetyada.
- 4.** Haddii aadan dooneyn inaad isticmaasho aabka moobeelka Tandem t:slim iyo Tandem Source, ama haddii aysan suurtagal ahayn inaad xogta la wadaagto rugtaada caafimaadka iyada oo loo marayo Tandem Source, waxaad isticmaali kartaa **Glooko**. beddelkeeda. Barnaamijkaan wuxuu ku siinayaa dulmar cad oo ku saabsan sida bamka u shaqeeyo iyo sida daaweyntaadu u egtahay. Waxaad isticmaali kartaa fiilo si aad bamka ugu xirto kumbiyuutar si aad u gasho my.glooko.com waana ay fududahay inaad xogtaada la wadaagto kooxdaada daryeelka. Macluumaadka waxaa loo soo bandhigay si kooban oo si fudud loo akhrin karo, kaas oo aad sidoo kale laga daawan karto aabka Glooko.



Habka ugu wanaagsan ee aad bamkaaga u jaajar gareyn karto

Sida ugu habboon, baytariga ku jira bamka insuliinta waa in loo jaajar gareeyo si gaaban. Ha jaajar gareyn bamka habeenkii inta aad huruddo. Waxaan kugula talineynaa inaad batteriga jaajar gareyso 10-15 daqiiqo maalintii. Jaajar gareynta batariga maalin kasta waxay la macno tahay inaad u baahnayn inaad jaajar gareyso muddo dheer hal mar, waxayna sidoo kale yareyneysaa khatarta ah in baytarigu dhammaado. Hal talo ayaa ah inaad fursad u hesho inaad bamkaaga jaajar gareyso intaad qubeysanayso, aad baabuur wado, ama aad hor fadhido TV-ga ama kumbiyuutarka.




Habka ugu wanaagsan ee aad bamkaaga u jaajar gareyn karto.



Jaajar garee muddo 15 daqiiqo inta lagu jiro maalintii.




Ka faa'ideyso aad u heli karto qubeyska.

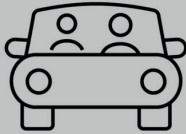

Jaajar garee inta aad gaari wado.




Jaajar garee inta aad daawanayso TV-ga.

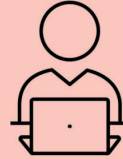

Jaajar garee inta aad fadhido kumbiyuutarkaaga.

Sidee baad u dooneysaa inaad bamkaaga u xidhato?

Waxaa jira dhowr qaabood oo ku habboon oo aad ku qaadi karto bamka insuliintaada. Waxaad dooran kartaa inaad ku dhejiso suunka, tusaale ahaan, ama aad ku haysato boorso yar oo dhexda ah. Wax badan ka ogow fursadaha kala duwan iyo sida looga dalbado alaabta [rubinmedical.se/produkt/barsystem/](https://www.rubinmedical.se/produkt/barsystem/)

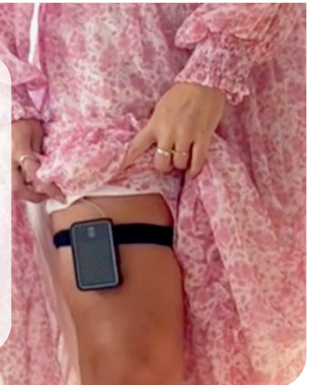

**Qiyaasta Bolus ee laga helo aabka moobeelka ee Tandem t:slim waxay u baahan tahay qaab taleefan oo casri ah oo la jaan qaadi kara iyo nidaam hawlgal oo la jaan qaadi kara. Cusboonaysiinta software-ka ayaa sidoo kale loo baahan doonaa in lagu sameeyo bamka insuliinta Tandem t:slim X2.*

Macluumaad muhiim ah oo ku saabsan badbaadada: Bamka insuliinta t:slim X2 waxaa loogu talagalay in lagu keeno insuliinta hoosaadka, iyadoo lagu saleynayo heerarka la dejiyay iyo kuwa isbadelaya, si loogu maareeyo cudurka macaanka dadka u baahan insuliinta. Bamka ayaa awood u leh inuu si kalsooni leh oo ammaan ah ula xiriiro aaladaha iswaafaqsan, ee oonleenka iskugu xirma.

Tiknoolajiyadda Control-IQ waxaa loogu talagalay in lagu isticmaalo kormeere gulukoos oo joogto ah (CGM) oo la jaanqaadaya iyo bamka insuliinta t:slim X2 si ay si toos ah u kordhiyaan, u yareeyaan, una joojiyaan bixinta insuliinta aasaasiga ah iyadoo lagu salaynayo akhrinta CGM iyo tirada gulukooska ee la saadaaliyay. Waxay sidoo kale keeni kartaa bolus-ka sixitaanka marka tirada gulukooska la saadaaliyo inuu ka badan yahay heerka hore loo qeexay. Bamka waxaa loogu talagalay in loo isticmaalo dadka da'doodu tahay lix sano iyo ka weyn kuwaas oo u baahan qiyaasta insuliinta maalinlaha ah ee ugu yaraan 10 unug ah oo miisaankoodu ugu yaraan yahay 25 kiilo garaam. Bamka waxaa loogu talagalay in hal bukaan loo isticmaalo. Ka eeg Hagaha Isticmaalaha liiska insuliinta la jaanqaadi karta.

DIGNIIN: Tiknoolajiyadda Control-IQ waa inaan loo isticmaalin qof ka yar 6 sano. Sidoo kale waa inaan loo isticmaalin dadka isticmaala ee u baahan wax ka yar 10 unug oo insuliin ah maalintii ama kuwa miisaankoodu ka yar yahay 25 kiilo garaam.

Nidaamkaan looma tilmaamin in loo isticmaalo haweenka uurka leh, dadka qaata keluo dhaqista, ama dadka aadka u xanuunsan. Ha isticmaalin nidaamka haddii aad isticmaalaysa hydroxyurea. Isticmaalayaasha bamka iyo nidaamku waa inay: diyaar u ahaadaan oo awoodaan inay isticmaalaan bamka insuliinta, CGM, iyo dhammaan qaybaha kale ee nidaamka si waafaqsan tilmaamaha isticmaalka; tijaabiyaan heerarka gulukooska dhiigga sida uu ku taliyay dhakhtarka daryeelka caafimaadkooda; muujiyaan xirfado ku filan oo tirinta karbohaydraytyada ah la xariira; ilaaliyaan xirfado is-daryeelid oo ku filan oo cudurka macaanka ah; si joogto ah ula kulmaan dhakhaatiirta daryeelka caafimaadka; iyo inay yeeshaan aragti iyo/ama maqal ku filan si ay u aqoonsadaan dhammaan hawlaha bamka, oo ay ku jiraan digniinaha, alaarmiyada, iyo xasuusinada. Haddii akhrinta CGM aysan la mid ahayn calaamadahaaga ama filashooyinkaaga, ka hubi sonkorta dhiiggaaga mitirka gulukooska dhiigga si aad u gaarto go'aanno ku saabsan maaraynta cudurka macaanka. Bamka t:slim X2 waa in laga saaraa kahor daaweynta MRI, CT, ama diathermy. Booqotandemdiabetes.com/safetyinfo si aad u hesho macluumaad dheeraad ah oo muhiim ah oo ku saabsan badbaadada.

Macluumaadka ka muuqda shaashadda aabka moobeelka ee Tandem t:slim daruuri ma ahan inuu la mid noqdo macluumaadka ka muuqda shaashadda bamka. Soo gelinta wireless-ka ee laga soo geliyo aabka moobeelka Tandem t:slim una gudubta madal Tandem Source oo ku salaysan oonleenka waxay u baahan tahay taleefan casri ah oo la jaan qaadi kara iyo isku xirka xogta wireless-ka ama shabakadda. Shaqaalaha daryeelka caafimaadku waa inaysan u isticmaalin soo dejinta barnaamijka Tandem Source si ay ula socdaan meelaha fog maadaama soo dejinta aysan dhicin waqtiga dhabta ah. Kharashyada xogta caadiga ah ee hawladeenka shabakada taleefankaaga gacanta ayaa laga yaabaa inay khuseeyaan.

Aabka moobeelka ee Tandem t:slim X2: Aabka moobeelka ee Tandem t:slim waxaa loogu talagalay inuu bixiyo shaashad labaad oo muraayad u ah bamka insuliinta Tandem t:slim X2 isla markaana xogta bamka ku soo gelin kara goobta Tandem Source si bilaa-waayir ah. Aabka Tandem t:slim looguma talgalin in loo isticmaalo in lagu xakameeyo bamka insuliinta ama lagu badelo macluumaadka lagu soo bandhigay bamka insuliinta. Go'aannada qiyaasta laguma sameyn karo iyadoo lagu saleynayo macluumaadka shaashadda aabka. Isticmaaluhu waa inuu raaca tilmaamaha ku qoran CGM iyo bamka insuliinta. Shaashada labaad ee uu aabku bixiyo looguma talgalin inay badesho is-kormeerka iyo daryeelka sida uu dhakhtarku ku taliyay.